Sinuskopie  
*Sinuscopy*

Sinuscopie  
*Sinuscopía*  
Sinuscopia

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**Particular attributes**

Due to our long experience, our scopes meet all technical requirements of a first class product:

- **compact objectives**  
the new generation of compact objectives used in our endoscopes provides the guarantee of full autoclavability even for a long period and gives the user the security of a long life of the product.
- **rod lens system**  
the new evaluated rod lens systems guarantees an illumination which really is second to none.
- **large image size**  
a high enlargement guarantees a comfortable working, even if the monitor is not so close to the operation field.
- **distal and proximal saphire windows**
- **comfortable direction of view on all scopes according to the necessity of the operation**
- **brilliant illumination and crystal clear resolution**
- **true colour reflection and regular focus on the whole image**
- **optimal depth of field, also according to the necessity of the operation**

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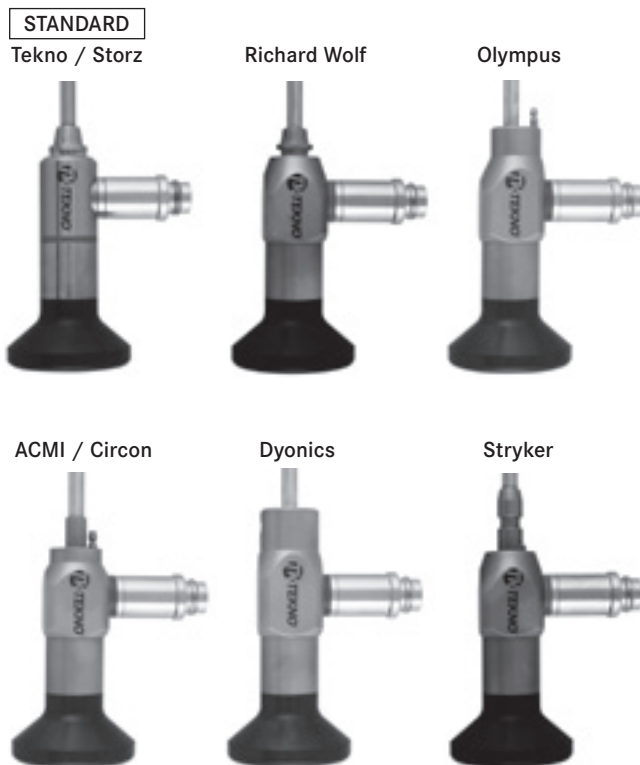
*In addition to our extensive program of endoscopes, we can manufacture endoscopes compatible to the following brands on demand.*

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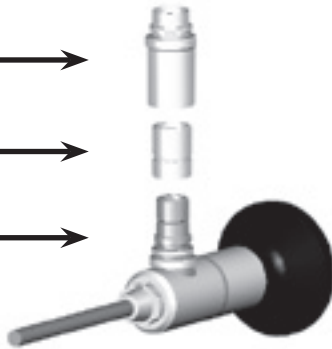
1. All autoclavable scopes are etched "autoclave 134°C/273°F 18 min".
2. All scopes may only be autoclaved at 134°C/273°F (±5°) for a maximum of 18 minutes. Failure to follow these parameters may destroy the scope.
3. Scopes should not be autoclaved at 120°C/248°F, as the prolonged sterilization time may damage the scopes.
4. The endoscopes must be thoroughly cleaned before autoclaving. Under no circumstances a hard object should be used for this purpose. We recommend the use of alcohol and cotton pads for the cleaning of optical surfaces (distal and proximal). Under no circumstances scopes may be cleaned with ultrasonic waves as this may also destroy them! Poorly cleaned surfaces lead to significant impairment and to dimming of the lenses.
5. The pre-cleaned scopes are placed in sterilization trays and containers or sterilization clothes specially designed for this purpose. Make sure to avoid any thermal bridge between the scopes and the sterilization container.
6. After the sterilization the scopes should cool slowly to room temperature while staying in the autoclave. Rapid cooling will destroy the scopes.
7. Please follow the instruction manual of the sterilization equipment and keep it controlled from time to time.

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
*Compatible light guide All products with this sign are fully autoclavable. We guarantee 24 months of full autoclavability for these scopes. (See instructions for autoclaving)*

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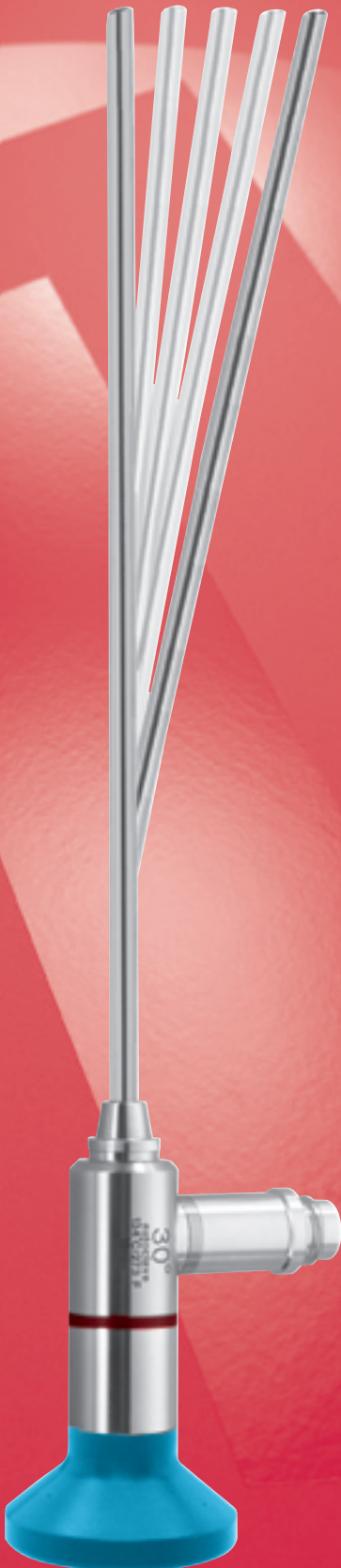## The advantages of Tekno-Flex endoscopes (patented):

- *the optical system is less sensitive to bending*
- *higher flexibility compared to standard rigid endoscopes*
- *higher robustness (70% of all repairing is due to broken rod-lenses caused by bent tubes)*
- *scratch-proof sapphire glass on the distal end and ocular side*
- *high resolution and contrast reproduction*
- *1 year warranty against broken rod-lenses*
- *Tekno-Flex scopes can be recognized by the suffix "SF"*

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